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UK:
Before first use
charge for 24 hours

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UK:
Insert the attachment
vertically

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UK:
Green LEDs: The device is
working correctly.
Red LED: The device needs to
be charged or the attachment
is defective.

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UK:
Replace attachments
every 3 months

Here you will find the most common problems that could occur with your emmi®-device, but which you can rectify yourself:

Flashing of the red LED means:

1. Battery discharged: Place the handpiece on the charging station so the emmi®-ultrasound device can start the charging process.

2. Indication of a defective attachment: Place a new attachment on the handpiece and switch the device on again. The green LED should now light up continuously.

Green LED lights up and turns off immediately:

1. No attachment attached: Please attach an attachment to the handpiece and switch it on again. The green LED should now light up continuously.

2. Attachment defective: Please attach a new attachment to the emmi®-ultrasonic device and try to switch it on afterwards. The green LED should now light up continuously.

The green LED lights up continuously, but without vibration:

Vibration was manually switched off: Please place an attachment on your emmi®-device. Press and hold the ON/OFF key for approx. 10 seconds to switch on. The LED lights up green three times. You can now remove your finger from the ON/OFF key. After switching the device on, the vibration should work again. To switch off the vibration, please proceed in the same way. The LED will light up red three times.

(Please note that our models emmi®-dent Platinum Care and emmi®-pet do not have a vibration motor and therefore this function cannot be used with these models).