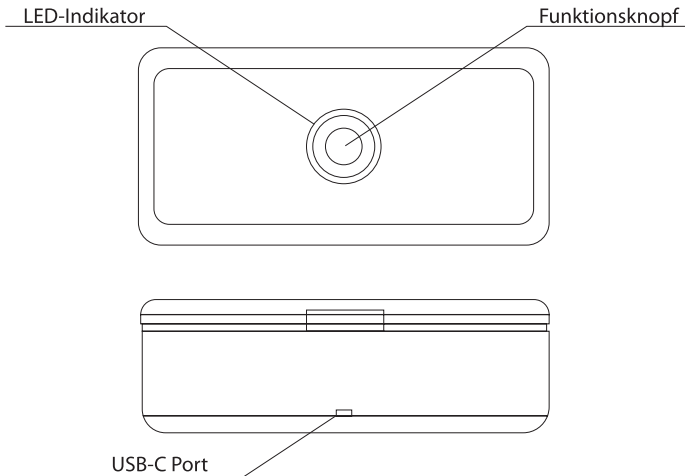
Einführung

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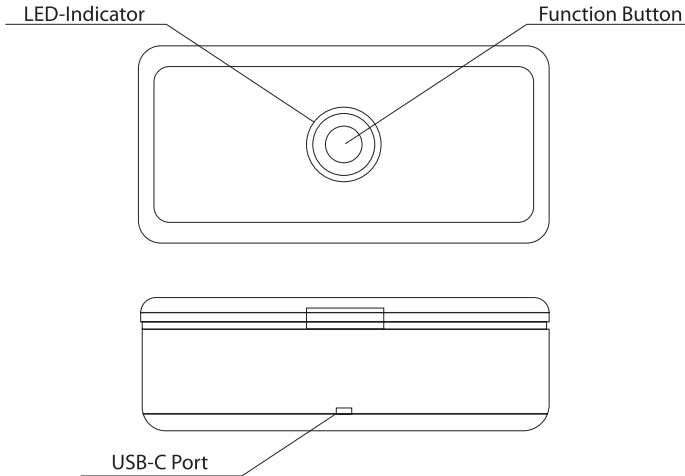
Introduction

emmi®-Steri UVC LED is a storage box with sterilization function(for hair-dressing, beauty, nail tools). The product applies medical grade 3 minutes sterilization, with sterilization rate up to 99,99%.

Instruction

1. Power on/off:

Press the button 3 seconds to start the device.
White light flashes once for standby mode.
Press the button 3 seconds to power off.

2. Sterilizing:

Pressing again to initiate 3 min sterilization during standby mode.
The LED-Indicator blinks white during the sterilization process.
The device will turn back to standby mode after sterilization.

3. LED-Indicator:

- (1) Start: white light and flashes once
- (2) Standby mode: white permanent light
- (3) Sterilizing: white blinking light

4. Automatic shutdown:

The device will turn off automatically after 15 hours of standby time.

5. Safety sensor:

A built-in sensor will turn off the UVC LED when the device is opened during the sterilization process to prevent eye damage.

Technical Data

Parameter	Value	Unit
Length	235	mm
Width	108	mm
Height	75,5	mm
Power	max. 8	W
Power supply	10 V = 700mA	-
Wavelength	260 - 280	nm
Input connector	Type-C USB	
Timer	3	min

Technical changes reserved.

UVC-Power standards for disinfection

Bacteria/Virus	UVC Power (mW)
Bacillus Coli	6,6
Influenza Virus	6,6
Bacillus Typhi	4,1
Bacillus Dysenteriae	4,2
Salmonella	7,6
Viralhepatitis	8,0
Micrococcus Gonococcus	8,5
Bacillus	10,0

UVC destroys the DNA of bacteria and kills all kinds of germs

Caution

1. Please read the manual carefully before using.
2. The working voltage has to be compatible with the product.
3. Do not stare at the UVC Light. It may cause discomfort or retinal damage
4. Keep the product away from children
5. Keep the device dry.
6. Please turn off the device to save power when not in use.
7. Bracket made of photosensitive material will change color if irradiated. It will recover its original color after a while.

emmi®-Steri UVC LED

Manual

99,99 % of Bacteria and Germs are destroyed
Sterilization in 3 min
turns of automatically after 15 hours in Standby
big variety of cleanable objects
simple use

Thank you for choosing our device. To avoid operating errors, please read the operating manual carefully.

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Emmi Ultrasonic GmbH